

Recommended Cleaners & Disinfectants

OAKWORKS®

NOTE: Reference the Cleaning Process section in the Product Manuals for further instruction.

Cleaning and disinfecting an OAKWORKS product is a three part process.

1. Clean any visible soil off the surface.
2. Disinfect surface per manufacturer's recommended contact time.
3. Rinse off disinfectant with clean water and thoroughly dry with a clean cloth or towel.

RECOMMENDED:

Oakworks recommends the use of disposable "paper roll" over the exam table/chair.

RECOMMENDED CLEANERS: For TerraTouch Fabric

- Formula 409 (EPA No.: 67619-10)
- Fantastik (EPA No.: 89900-3)
- Windex Multisurface (EPA No.: 4622-593)
- Force of Nature (EPA No.: 93040-1)

DISINFECTANTS:

- Formula 409 (EPA No.: 67619-10)
- Parker Labs Protex (EPA No.: 6836-152)
- Madacide-FD (EPA No.: 1130-15)
- Clorox EZ Kill Wipes (EPA No.: 5984-10)
- Clorox Disinfecting Wipes (EPA No.: 67619-9)
- Clorox Healthcare VersaSure Wipes (EPA No.: 67619-37)
- Clorox Healthcare Fuzion (EPA No.: 67619-30)
- PureGreen 24 (EPA No.: 72977-3)*
- Rejuvenate Disinfectant Wipes (EPA No.: 74559-3)
- Force of Nature (EPA No.: 93040-1)
- Kurland Fog It IQ
- Sani Wipe AF3 (EPA# 9480-9)
- Sani Wipe Super (EPA# 9480-4)
- Sani Wipe Bleach (EPA# 9480-8)

**This list supersedes all previous versions.
Recommended for upholstery only.*

NOT RECOMMENDED:

- Citrus based cleaners
- Acetone
- Hydrogen Peroxide based cleaners/disinfectants
- Alcohol-based cleaners or disinfectants
- Acetic Acid
- Phenolic
- Bleach
- Unapproved quaternary ammonias ("quats")

RECOMMENDED CLEANERS: For Ultraleather® Fabric

To maintain the beauty and longevity of your Ultraleather upholstery, we recommend using only approved cleaners and disinfectants.

For a full list of tested and approved products, please visit the [Ultrafabrics Care & Cleaning Guide](#).

Note: All cleaners and disinfectants have the ability to degrade and damage the upholstery. This information is not a guarantee, however, following the recommended cleaner and disinfectant list and cleaning process will provided the best care for your OAKWORKS products. The use of other cleaning agents or disinfectants is not recommended and may void OAKWORKS warranty.

If you have any questions, call Customer Service at 717.235.6807